Bruksanvisning

Fingertupppulsoksymeter

XM-113

Dokumentnr.:JDXM-0304-055
Version: D
Utsædatsdato: 2025.04.21

Generell beskrivelse

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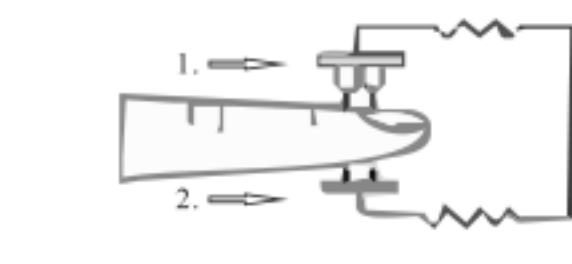
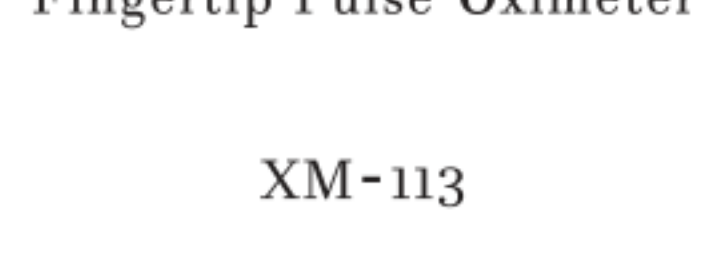
Document No.: JDXM-0304-055 Version: D
Date of Issue: 2025.04.21

General Description

Oxygen binds to hemoglobin in red blood cells when moving through the lungs. It is transported through the body as arterial blood. A pulse oximeter uses two frequencies of light (red and infrared) to determine the percentage(%of hemoglobin in the blood that is saturated with oxygen. The percentage is called blood oxygen saturation, or SpO2. A pulse oximeter also measures and displays the pulse rate at the same time it measures the SpO2 level. This device conforms to IEC60601-1, IEC60601-1-2, IEC60601-1-11, ISO 80601-2-61.

PRINCIPLE of the oximeter is as follows: The pulse oximeter works by applying a pulsating arterial vascular bed. The sensor contains a dual light source and photo detector. The one wavelength of light source is 660nm, which is red light the other is 905nm, which is infrared-red light skin, bone, tissue and venous vessels normally absorb a constant amount of light over time. The photo detector in finger sensor collects and converts the light into electronic signal which is proportional to the light intensity. The arterial bed normally pulsates and absorbs variable amounts of light during systole and diastole, as blood volume increases and decreases. The pulse rate at the same time it measures the SpO2 level.


This device conforms to IEC60601-1, IEC60601-1-2, IEC60601-1-11, ISO 80601-2-61.

PRINCIPLE of the oximeter is as follows: The pulse oximeter works by applying a pulsating arterial vascular bed. The sensor contains a dual light source and photo detector. The one wavelength of light source is 660nm, which is red light the other is 905nm, which is infrared-red light skin, bone, tissue and venous vessels normally absorb a constant amount of light over time. The photo detector in finger sensor collects and converts the light into electronic signal which is proportional to the light intensity. The arterial bed normally pulsates and absorbs variable amounts of light during systole and diastole, as blood volume increases and decreases. The pulse rate at the same time it measures the SpO2 level.

This device conforms to IEC60601-1, IEC60601-1-2, IEC60601-1-11, ISO 80601-2-61.

Diagram of Operation Principle

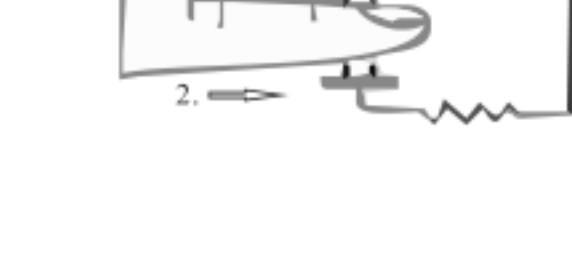- Red and infrared-ray emission tube
- Red and infrared-ray receipt tube

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
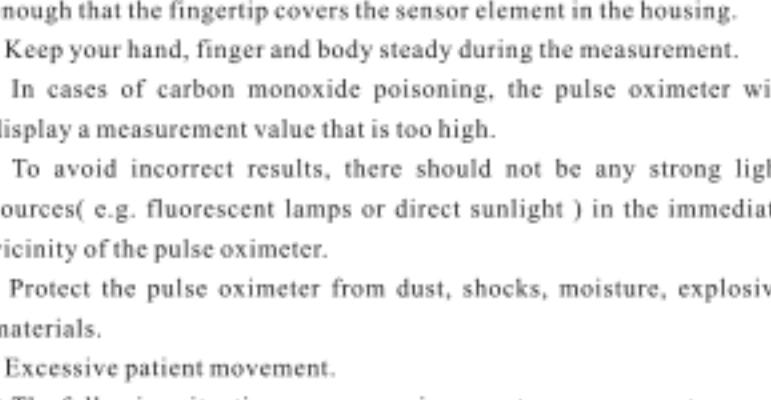
Monitorenet

Display

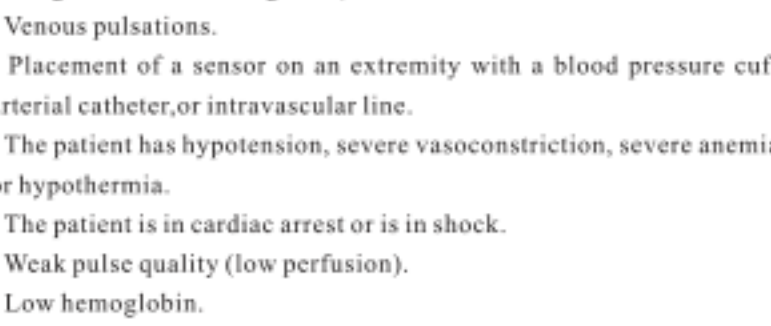Intended purpose/Indications/Contraindications

- Non-observance of the following instructions can lead to incorrect or failed measurements

- There must not be any nail polish, artificial nails or other cosmetics on the finger to be measured.

- Ensure that the finger nail on the finger to be measured is short enough so that the fingerclip covers the sensor element in the housing.

- Keep your hand, finger and body steady during the measurement.

- In cases of carbon monoxide poisoning, the pulse oximeter will display a measurement value that is too high.

- To avoid incorrect results, there should not be any strong light sources (e.g. fluorescent lamps or direct sunlight) in the immediate vicinity of the pulse oximeter.

- Protect the pulse oximeter from dust, shocks, moisture, explosive materials.

- Excessive patient movement.

- The following situations may cause inaccurate measurements

- Significant levels of dysfunctional hemoglobin (such as carboxyl hemoglobin or methemoglobin).

- Venous pulsations.

- Placement of a sensor on an extremity with a blood pressure cuff, arterial catheter, or intravascular line.

- The patient has hypotension, severe vasoconstriction, severe anemia, or hypothermia.

- The patient is in cardiac arrest or is in shock.

- Weak pulse quality (low perfusion).

- Low hemoglobin.

Unit Illustration

- Indications for use/Intended Use:The fingertip pulse oximeter is a portable non-invasive,spot-check-oxygen saturation of arterial hemoglobin and pulse rate of adult at home and/hospital (including clinical use in intensive-care,anaesthesia etc).

- Medical indications:Oxygen saturation normal and hypoxemia.

- Name of disease or condition:Well-being and the disease which may be result in hypoxemia.

- Patient population:Adults.

- Intended user:Professionals and Lay person.

- Contraindications:The device is not for continuously monitoring.

Package contents

- 1 x XM113 pulse oximeter

- 1 x Owner's Manual

- 1 x Retaining strap

- Optional accessories

- 2 x 1.5v AAA batteries

Monitor Unit

System Settings

With power off, Press the power button about 2 seconds to activate system setting.

Settings available for Typ, Beep, Language, Default, SpO2 Typ Lo, PR Typ Hi, PR Typ Lo and EXIT. Long press to enter the specific value setting, short press to exit among the setting items.

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